




# EXTERNAL FIXATION TGF

Back to basics,  
simplicity, stability  
and versatility

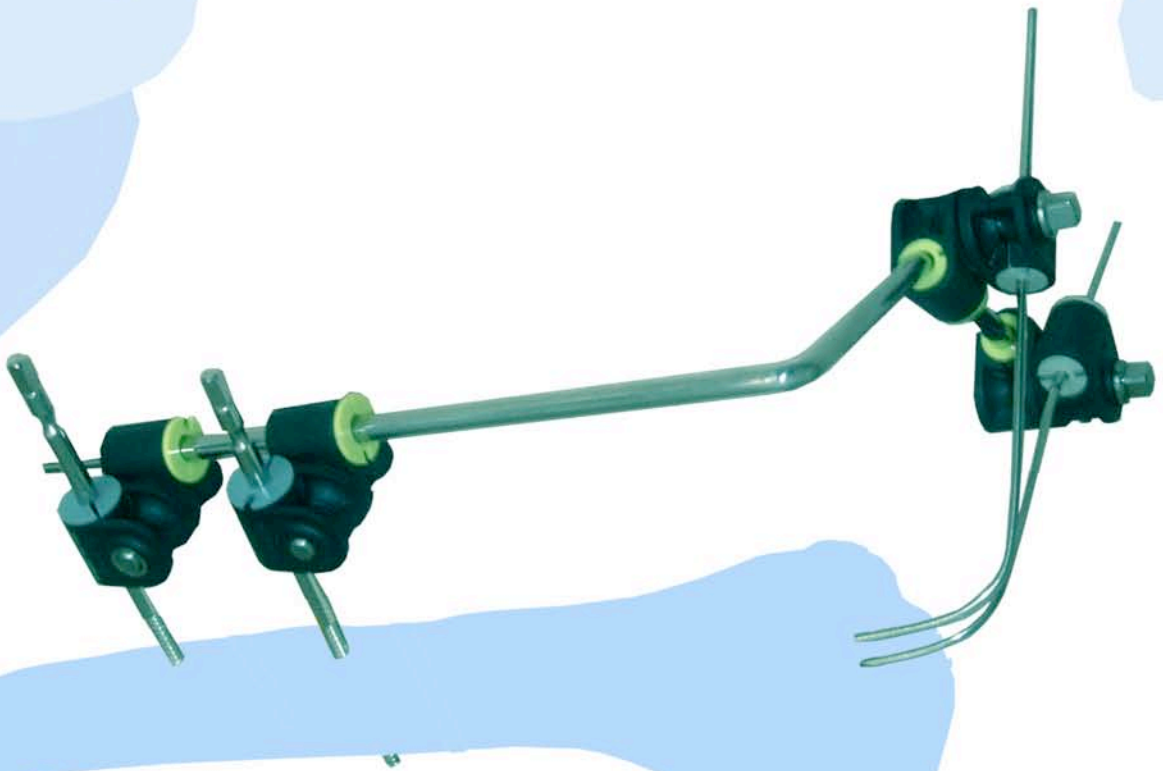


IT'S GOT TO MAKE SENSE







# THE GEXFIX EXFIX SYSTEM


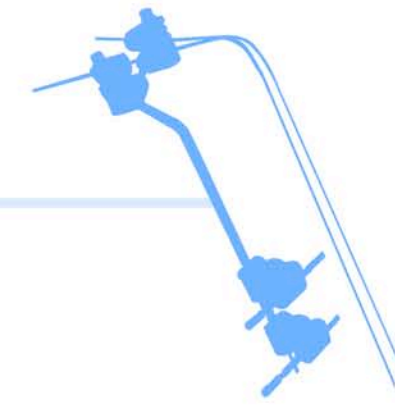


## THE TENSION GUIDE FIXATOR :


- The tension Guide Fixator system (TGF), Kit for the superior limb fractures, is based in two wires guides that are tensioned as needed for the reduction.
- Since we started using the TGF system, we have no longer needed to use any other method of treatment for the surgical fractures of the Humerus neck.
- With this method and with this system, it's also possible to treat Humerus neck fractures in a simple way.

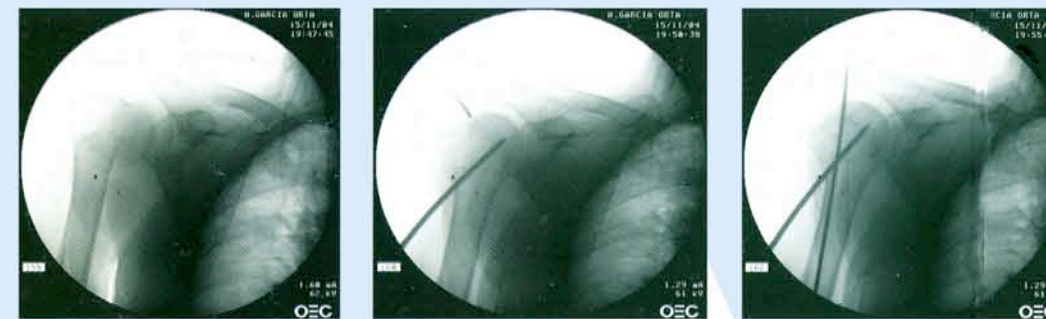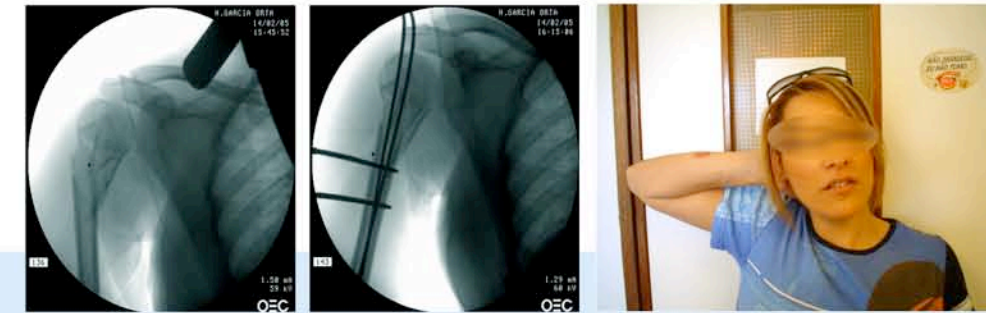

- 1 Intent of reduction by external manipulation.
- 2 Using the 5 mm pin for reduction and introduction of the wire guide.
- 3 Putting in place the first K.wire guide.

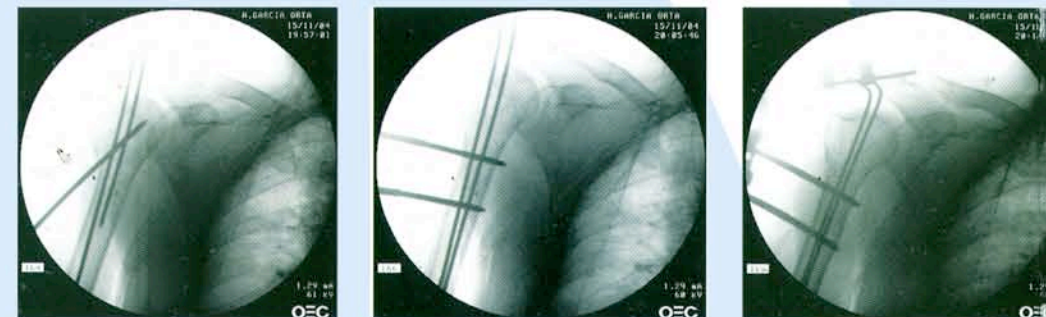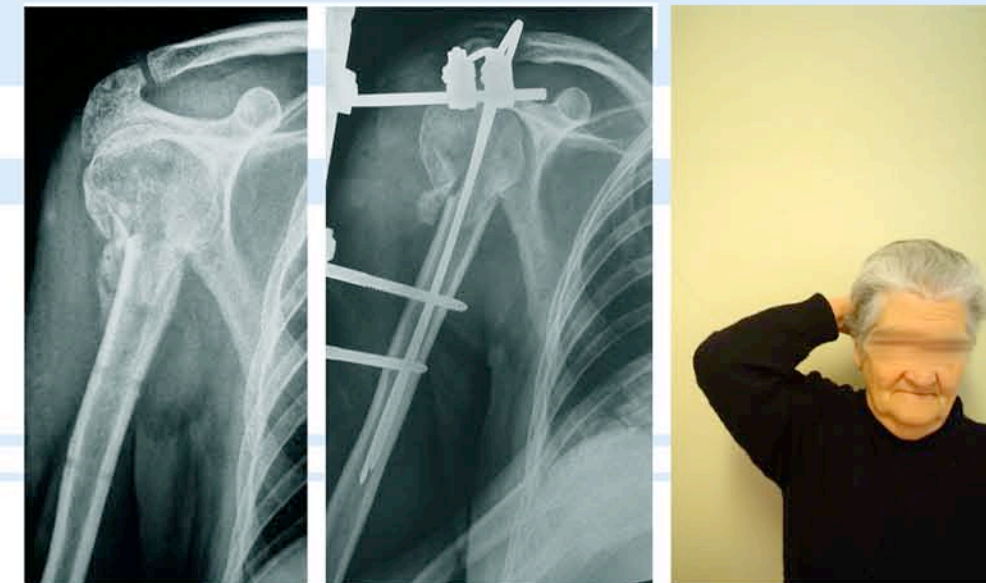

- 4 Putting in place a second K.wire guide to control the rotation.
- 5 Putting in place the self drilling, self tapping pins after of drilling the diaphisis.
- 6 Tensioning the K.wire guides and put in place the TGF system.

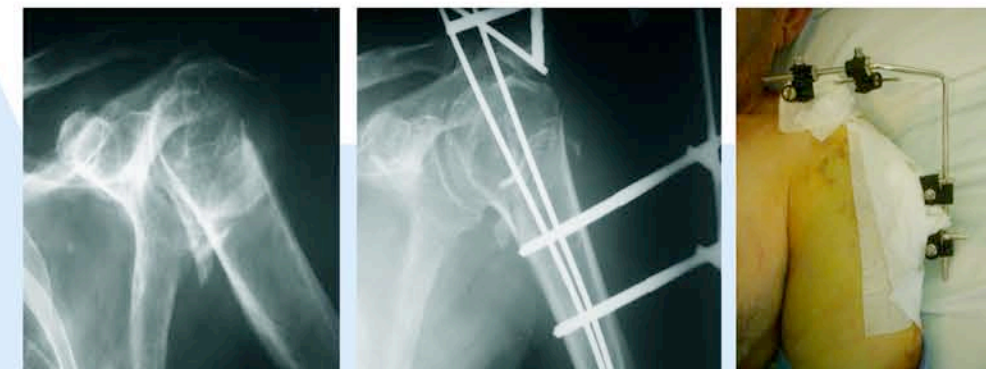




**COMPONENTS**

**DESCRIPTION**

**REFERENCE**

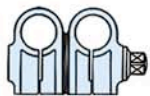

**Connection**  
diameter 9mm/9mm

1010-1040



**Reduction Sleeve**  
diameter 9mm/2,5mm  
diameter 9mm/4.0mm  
diameter 9mm/5mm

1010-1055  
1010-1053  
1010-1054



**Double-L Bar**  
left rod diameter 5mm  
right rod diameter 5mm

1010-1081  
1010-1082



**Carbon rod**  
diameter 9mm, length 110mm  
diameter 9mm, length 150mm

1010-1064  
1010-1065



**Lengthening /compression tube**

1010-1100



**Kirschner wire**  
diameter 2,5mm, length 300mm  
diameter 2,5mm, length 300mm, 25 mm thread middle  
diameter 2,5mm, length 250mm tip threaded

1020-1032  
1020-1033  
1020-1034



**Self drilling /self tapping pin**  
Pin diameter 4mm, length 100mm, thread length 40mm  
Pin diameter 4mm, length 145mm, thread length 45mm  
Pin diameter 4mm, length 180mm, thread length 50mm  
Pin diameter 5mm, length 210mm, thread length 55mm

2015-1020  
2015-1025  
2015-1030  
2020-1030



**Trocar and sleeve**  
diameter 5mm, length 70mm  
diameter 5mm, length 120mm

5025-1000  
5025-1200



**Handle for trocar and sleeve**

5025-1100



**Screw handle**

5015-1000



**Drill brace**

5010-1000



**Sterilisation Tray**

6010-1007

Distributed by :

CE 0459  
EN 93/42  
FDA



Cat. No: TGF 05-2006

To ensure the best quality of its products and their improvements,  
Gexfix reserves the right to modify all or part of their products



Gexfix S.A.  
Av. De La Praille 50  
CH-1227 Carouge  
Tél : +41 22 308 16 70  
Fax : +41 22 308 16 79  
www.gexfix.ch  
ISO 9001/2000  
ISO 13485/2003